





# Video Laryngoscope File Upload Procedure


1. Plug camera into ePCR tablet and launch the Airtraq Cam application on the desktop

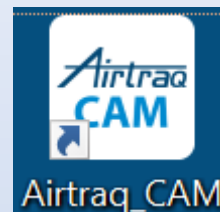



2. Close this window if it appears

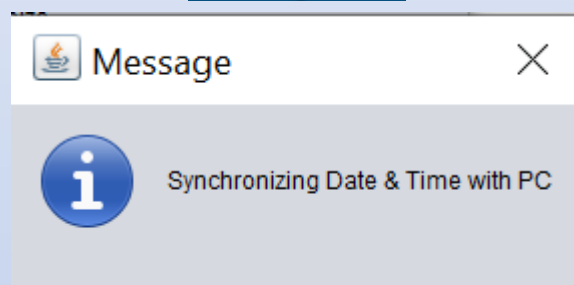

3. Answer "Yes" if it asks you to update the software

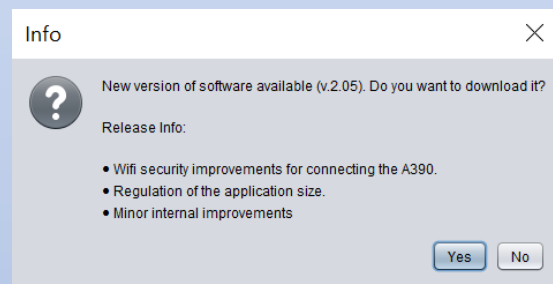

4. When the software launches, choose "Download Videos & Snapshots to PC"

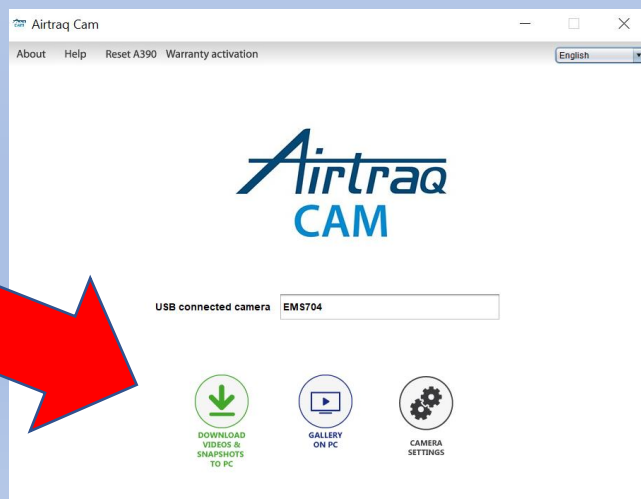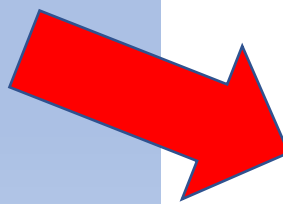




# Video Laryngoscope File Upload Procedure p.2

5. Choose the correct file from the available list and click the green down arrow here


Camera ID: EMS704

0 videos , 0 GB/ 8 GB (8 GB free space)

| Date                 | Duration | Downloaded to PC    | Size       |
|----------------------|----------|---------------------|------------|
| 2019-08-21 16h28m12s | 05:49    | 2020/07/30 09:45:28 | 61.9 MB    |
| 2019-08-21 17h20m38s | 00:00    | NO                  | 3.8 kB     |
| 2019-08-21 20h23m00s | 05:51    | NO                  | 37.5 MB    |
| 2019-08-21 20h31m52s | 03:48    | NO                  | 38.9 MB    |
| 2019-08-21 20h36m00s | 01:00    | NO                  | 52.9 MB    |
| 2019-08-21 20h36m01s | 00:05    | NO                  | 713.9 kB   |
| 2019-08-24 13h41m23s | 02:08    | NO                  | 15.5 MB    |
| 2019-08-24 13h43m48s | 00:08    | NO                  | 1,014.8 kB |

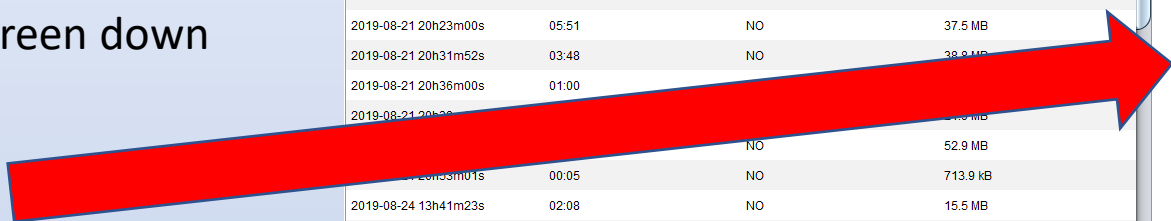


6. On the desktop, open the "VL Uploader"

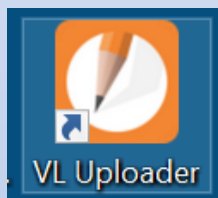

7. Enter the incident number and click on the file selector

8. Select the correct file from the airtraq>gallery folder

*This PC/documents/airtraq/gallery*

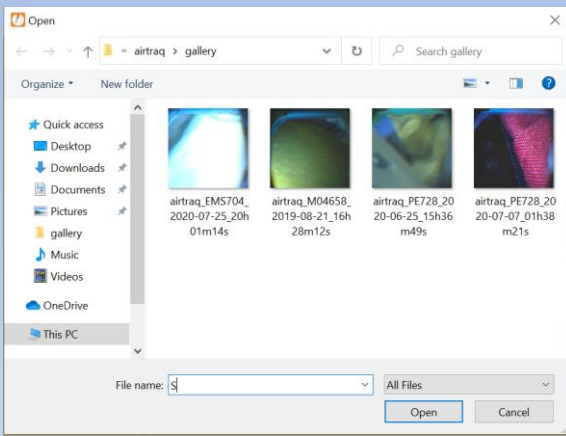

9. Submit the form – attachment to eMEDS will occur centrally

